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## Cleanses eyelids affected by pathology

Gel containing 0.2 % hyaluronic acid (Sodium hyaluronate)  
Ophthalmologically tested

0 % preservatives. 0 % parabens. 0 % perfume. 0 % alcohol. 0 % lanolin.  
Epifree Technology is an exclusive and patented manufacturing process.  
It guarantees absence of all preservatives and maintains the integrity of the active ingredients.

**PRESENTATION:**  
50 ml bottle with airless pump

**COMPOSITION:**  
0.2% Sodium hyaluronate, Allantoin, Polysorbate 20.  
Other ingredients:  
Water, Glycerin, Hydroxyethyl acrylate / Sodium acryloyl dimethyl taurate copolymer, Citric acid.

**WHEN SHOULD ILAST® HYDRACLEAN BE USED?**  
Gel intended for cleansing unhealthy eyelids, especially the lid margin. For use in all types of blepharitis: seborrhoic dermatitis, meibomitis, etc.  
Ilast HydraClean mechanically removes flaky skin, sebum deposits, secretions and crusts from the eyelids and the roots of the eyelashes.  
The combination of sodium hyaluronate and allantoin helps to maintain hydration and encourages skin healing.

**INSTRUCTIONS FOR USING ILAST® HYDRACLEAN**  
- Turn the top of the bottle anti-clockwise (see the arrow on the pump) while holding the bottle in the other hand. The pump is then unlocked.  
- When using for the first time, push several times on the top of the bottle to prime the pump.  
- Soak a pad in ILAST HydraClean gel. Keep the eye closed and apply the pad gently to the eyelid and the base of the lashes.  
- Rub the lid gently with circular movements.  
- Using another pad, repeat this sequence for the other eyelid.  
- After use, lock the pump by turning the top of the bottle clockwise.  
- If the pump becomes unprimed during use, push several times on the top of the bottle to reset it.  
- It is not necessary to rinse after each use.  
- Cutaneous route.

**WARNINGS:**  
• Do not swallow.  
• Keep out of the reach of children.  
• Do not use after the expiry date shown on the bottle.  
• Do not open the bottle (risk of bacterial contamination).  
• Do not use if the system shows signs of wear: damage to the bottle, pump disconnected, etc. (risk of bacterial contamination).  
• Use within 3 months of opening.  
• Store at a temperature between 2 and 35 °C.

**CONTRAINDICATIONS:**  
Allergy to one of the ingredients.

**WHAT IS EPIFREE TECHNOLOGY®?**  
HORUS PHARMA laboratories have developed Epifree Technology®. This is an exclusive and patented manufacturing process, which guarantees absence of preservative agents, while maintaining the integrity of the active ingredients. Thus, neither hyaluronic acid, which tolerates standard decontamination processes poorly, nor the other active ingredients are affected.  
The eyelids are a particularly sensitive and vulnerable part of the body. Closeness to the eye precludes the use of common skin treatments.  
As a result of the progress made by Epifree Technology®, the latest generation from the ILAST® product range is particularly suitable for cleansing and caring for skin that is highly sensitive, allergic or affected by a medical disorder.

**CE 1014**  
Medical device complies with the requirements of directive 93/42/EEC.  
Date of brand application (01/2008; rev 2012)

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