GEBRAUCHSINFORMATION

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INSTRUCTIONS FOR USE

Aktimed® TAPE PLUS is an elastic self-adhesive tape bandage (kinesiology tape) with an integrated highly purified plant extract from Arnica montana, Rhus toxicodendron and Ruta graveolens.

The main effects of the traditional kinesiological tape are supported by the properties of the plant extracts, providing pain relief, reducing swelling and promoting blood circulation. Based on the interaction with the tape material, the plant extract is transferred efficient and effectively into the damaged structure. Therefore, the additional use of oral medicine or cream to relieve pain and/or reduce swelling may possibly be decreased.

APPLICATION AREAS

- Prophylaxis
- Therapy
- Rehabilitation of muscle-, tendon- and joint-complaints

Examples: Cervical spine syndrome, thoracic spine syndrome, lumbar spine syndrome, whiplash, recent injuries with pronounced hematomas, bruises, overexpansion, sacroiliac joint blockage, injuries at tendons and ligaments, tension headaches, rheumatic diseases, carpal tunnel-syndrome, tennis- and golf-elbow, rotator-cuff injury, impingement syndrome, migraine, gonarthrosis, coxarthrosis, scar treatment, nerve inflammation, hallux valgus, heel spur, lymphedema, fibrosis, sciatica, achilles tendon injury, post-treatment of cast immobilisations of fractures and dislocations

ACTION SPECTRUM

Regulation of muscle tone, improving muscle function, stabilization of joints, promoting mobility, pain relief, stimulating metabolic functions, promoting local blood circulation, activation of lymph systems, promoting microcirculation.

APPLICATION INSTRUCTIONS

- The product can only be used once and needs to be applied by a trained person only.
- Prior to application the skin should be cleaned and fat-free.
- Remove the protection film. Depending on the tape, apply with or without strain.
- The glue unfolds the adhesive effect through body temperature. For this purpose it is advised to smooth out the tape in longitudinal direction from base to end (approx. 60 sec.).
- Apply the tape 30 min prior to sports activity.
- After showering, taking a bath or swimming the tape should be dried with a hair dryer (lowest setting) or by dabbing.
- The tape can occasionally cause skin irritation. In that case please remove the tape.
- The tape can best be removed after taking a shower or a bath.

SAFETY INSTRUCTIONS

- Do not use in case of an allergy against one of the ingredients, or test the tape on a small area of skin first!
- Do not apply on open wounds, injured skin or unhealed scars!
- Do not use in case of acute or chronic skin diseases (e.g. neurodermatitis thrust, Psoriasis, fungal infection)!
- Do not use in case of Thrombose or Thrombophlebitis!
- Do not use in case of a peripheral arterial occlusive disease distal to the occlusion!
- Do not use in case of malignant tumors, radiotherapy or chemotherapy!
- If pregnant, please apply the tape only according to your doctor's advice!
- Keep out of reach of children!

DURATION OF USE

The tape lasts up to 6 days on the skin.

TECHNICAL INFO

AT 20100 Aktimed® TAPE PLUS (kinesiology tape with additional benefits)

Supporting material: 95% cotton, 5% spandex

Adhesive composition: synthetic rubber based adhesive mass
Ingredients: Arnica montana 10⁶, Rhus toxicodendron 10⁸, Ruta graveolens 10⁶

Size: 5 cm x 3 m

Store in a cool, dry place away from direct sunlight between 8°C and 25°C.

CE: Compliant EU Guideline 92/42/EWG for medical products

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